



AED INSTRUCTIONS


1 **TURN ON**

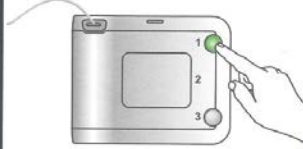


2 **Bare chest.**
Clean and dry if needed.




3 **Open gray pads case.**




4 **Peel each pad.**

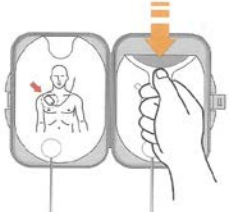


5 **Place pads on chest.**

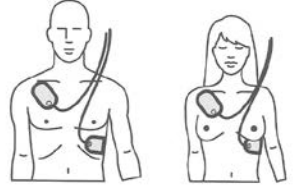


6 **Analyzing ... Stand back**


WAIT UNTIL  STOPS FLASHING



7 **Press flashing orange shock button.**




8 **For CPR help: Press the flashing blue button.**




INFANT/CHILD
< 55 lbs / 25 kg (0-8 years)

Insert key at beginning of rescue.



INFANT/CHILD

Place pads on back and chest.

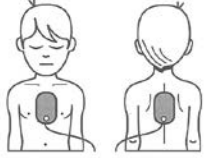

AED INSTRUCTIONS